

# roturner

3 YEAR WARRANTY

Manufactured and tested in Canada to the highest standards by **Jaide Products Inc.** The **Roturner Sit to Stand Transfer Aid's** warranty covers manufacturers defects for a period of three years from the date of original purchase.

During the warranty period **Jaide Products Inc.** will replace or repair, at no cost, any part of the **Roturner** which proves to be defective due to manufacture defects or poor workmanship.

### The warranty does not cover:

Wear and tear, improper usage, misuse or abuse, negligence, accident or alteration.

# roturner



 **Made in Canada**



32 Seapark Dr, St. Catharines,  
Ontario, CANADA L2M 6S6  
Tel: 289-968-5968  
Email: [info@jaideproducts.com](mailto:info@jaideproducts.com)  
Web: [jaideproducts.com](http://jaideproducts.com)

## INSTRUCTIONS FOR USE

# rotturner



## Caregivers and nurses face a high risk of back pain and musculoskeletal disorders (MSDs) due to frequent manual transfers.

The **Rotturner Sit-to-Stand Transfer Aid** is designed to reduce the need for manual lifting, helping protect caregivers while providing a safer, more comfortable, and dignified transfer experience for clients.

- **Sturdy & Stable:** A robust frame provides confidence and support when transitioning between a bed, chair, commode, or wheelchair.
- **Secure & Adjustable:** The wide turntable creates a stable standing surface and rotates into five locked positions for safe, controlled movement.
- **Built to Last:** Strong, durable, and designed for long-term use in any care setting.

Established in 2018, **Jaide Products Inc.** developed the Rotturner to meet the growing need for safe and practical transfer solutions in North America.

Proudly manufactured in St. Catharines, Ontario by Niagara Precision Limited, the Rotturner is a trusted, **Canadian-made** solution for safer caregiving.




## HOW IT WORKS

1

**ASSESS  
PATIENT CAN  
WEIGHT BEAR**

2

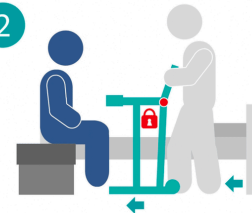

Before moving the **Rotturner** be sure it is in the locked position. The caregiver then wheels the **Rotturner** into position in front of the seated user.

Press the foot bar down and gently glide the **Rotturner** closer to the user and under their feet.

3



The user places their feet on the turntable and hands on the handle grip area. They are then assisted from the seated position to a standing position.


4



Lifting the brake handle the caregiver then unlocks and rotates the turntable left or right.

The caregiver then releases the brake into one of 5 red locking positions.

5

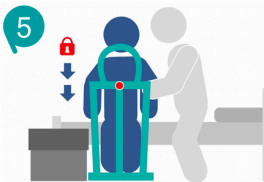

The caregiver may now assist the user to be seated on the previously placed wheelchair or other seat, then remove the **Rotturner**.



[www.jaideproducts.com](http://www.jaideproducts.com)