

# roturmer

## 3 YEAR WARRANTY

Manufactured and tested in Canada to the highest standards by **Jaide Products Inc.** the **roturmer Sit to Stand Transfer Aid's** warranty covers defects in materials or workmanship for a period of three years from the date of original purchase.

During the Warranty period **Jaide Products Inc.** will replace or repair, at no cost, any part of the **roturmer** which proves to be defective due to manufacture defects or poor workmanship.

If **Jaide Products Inc.** is unable to provide a replacement part or undertake a suitable repair or if such repair is impracticable or cannot be made in a timely fashion, **Jaide Products Inc.** reserves the right to replace the whole unit.

### This Warranty does not cover:

Wear and tear, improper usage, misuse or abuse, negligence, accident or alteration.

# roturmer



 **MADE IN CANADA**

## INSTRUCTIONS for use



32 Seapark Dr, St. Catharines,  
Ontario, CANADA L2M 6S6  
Tel: 289-968-5968  
Email: [info@jaideproducts.com](mailto:info@jaideproducts.com)  
Web: [JaideProducts.com](http://JaideProducts.com)

# rotturner



## The risk of back pain and other musculoskeletal disorders (MSDs) is a very real issue for caregivers and nurses.

The use of a **rotturner** sit to stand transfer aid can remove the need for manual handling lowering the risk of injury to the caregiver, while providing a safer, more dignified and comfortable transfer solution for the client.

The sturdy frame instills confidence in the client as they make the transition between a bed, seat, commode or wheelchair. The wide turntable provides a stable and safe area that the client can step onto while the transfer takes place.

It rotates between five different positions with a locking device to ensure client safety.

## Robust, strong and built to last!

Established in 2018 **Jaide Products Inc.** was created to bring a much needed client transfer aid to the North American Market.

The **rotturner** is manufactured in St. Catharines, Ontario by Niagara Precision Limited for **Jaide Products Inc.**



## How it works!



Before moving the rotturner be sure it is in the locked position. The assistant then wheels the **rotturner** into position in front of the seated client.

Press the black foot bar down and gently glide the **rotturner** closer to the client and under their feet.



The client places their feet on the turntable and hands onto the handle grip area and is assisted from the seated position to a standing position.



Lifting the brake handle the assistant then unlocks and rotates the turntable left or right.

The assistant then releases the brake into one of 5 locking positions.



The assistant may now transfer the client to a previously placed wheelchair or other seat, then remove the **rotturner**.

SEE OUR VIDEO DEMONSTRATION AT [rotturner.com](http://rotturner.com)